


**BLINCON DISPOSABLE COSMETIC CONTACT LENS
INSTRUCTION FOR USE**

Name	Type	Dia. (mm)	Curvature (mm)	Diopter (D)
Blincon	Disposable Cosmetic Contact Lens	14.0 ~ 15.0	8.5 ~ 8.8	-25.00 ~ 0.00

Indication for Use

Blincon Disposable Cosmetic Contact Lenses is for correct visual acuity in aphakic and non-aphakic persons with non-diseased eyes with myopia. It is also meant for cosmetic purpose.

Schedule for Daily Wear / Recommended Replacement Schedule

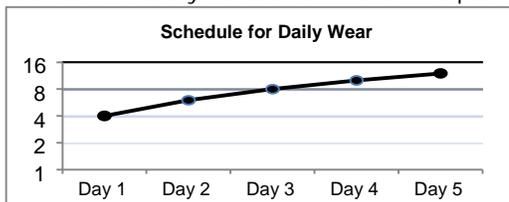

- The lenses are meant for daily wear.
- To be discarded and replaced with new lens every month.
- Always discard lenses worn on a frequent replacement schedule after the recommended wearing schedule by your Eye Care Practitioners.
- Never wear your lenses beyond the period recommended by your Eye Care Practitioners.

Before Use

To prevent infection, please take precautions as follow:

- Check the expired date of the lens before buying.
- Do not use if the sterile package is opened or damaged.
- Use recommended intensive cleaner to wash lens.

Lens Application

- Wash hands thoroughly with soap and dry them with a lint free towel. Rinse the lens thoroughly with multi-purpose solution.
- Place the lens on the tip of your index finger. Make sure the lens is in a bowl shape and correctly oriented.



- Hold the upper eyelid firmly with one finger of the other hand. (Fig 1)
- While holding down the lower eyelid with one finger of your applying hand, carefully place contact lens on your cornea. (Fig 2)
- Slowly release upper eyelid, gently close eye to let the lens set in place. Repeat same procedure for the other eye. (Fig 3)

Lens Removal

- Wash hands thoroughly with soap and dry with a lint free towel.
- Check the lens position on cornea.
- Hold down lower eyelid with middle finger.
- Use forefinger to slide lens to lower part of eye.
- Gently pinch the bottom edge of the lens with the pads of index finger and thumb to remove lens.

- Repeat the same procedure for the other eye.

Daily Cleaning and Care Regimen

- After removing the lens from the eye, place it on the palm of your hand and apply a few drops of multi-purpose solution onto the lens.
- Rub the lens gently with your index finger against the palm for about 20 seconds on both sides.
- Rinse the lens thoroughly with multi-purpose solution.
- Soak the clean lens in the case with multi-purpose solution for next use.
- Never rinse lens with tap water.

Contraindications/Warnings/Precautions

- Do not use the contact lens if any of the following conditions exist:
 - Eye disease, injury or abnormality that affects the cornea, conjunctiva, or eyelid
 - Any active corneal infection (bacterial, fungal, protozoal or viral)
 - Eye red or irritated
- Proper use and care of contact lens and lens care products, including lens cases, are essential for the safe use of these products.
- If you experience eye discomfort, excessive tearing, vision blur, redness of the eye or other problems, you should remove your lens immediately and contact your Eye Care Practitioner.
- Always keep your lens immersed in a recommended storage solution when lenses are not being worn (stored). Prolonged periods of drying will reduce the ability of the lens surface to return to a moist state.
- Ask your Eye Care Practitioner about wearing lenses during sporting activities.
- Put on lenses before putting on makeup. Do not get cosmetics, lotions, soaps, creams, deodorants, or sprays in eyes or lenses.
- Do not share contact lenses with others.

Symbol

	Date of Manufacture		Steam Sterilized
	Lot Number		Manufacturer
	Use By		EU Representative
	Catalog Number		Consult Instruction For Use

Manufacturer

Dreamcon Co., Ltd
10, Yeongdong 2-gil, Yangsan-si, Gyeongsangnam-do, Korea.
Tel: +82 (55) 381-0300 Fax: +82 (55) 366-0211

Principle One O One Vision Creations Sdn. Bhd. (830877)

2nd Floor, Sublot 11, Survey Lot 1529, Block 17,
Jln Lapangan Terbang, 93250Kuching Sarawak Malaysia.
Tel: +6 082-383 101 Fax: +6 082-383 102