


**BLINCON DISPOSABLE COSMETIC CONTACT LENS
INSTRUCTION FOR USE**

| Name | Type | Dia. (mm) | Curvature (mm) | Diopter (D) |
|---------|----------------------------------|-------------|----------------|---------------|
| Blincon | Disposable Cosmetic Contact Lens | 14.0 ~ 15.0 | 8.5 ~ 8.8 | -25.00 ~ 0.00 |

Indication for Use

Blincon Disposable Cosmetic Contact Lenses is for correct visual acuity in aphakic and non-aphakic persons with non-diseased eyes with myopia. It is also meant for cosmetic purpose.

Schedule for Daily Wear / Recommended Replacement Schedule

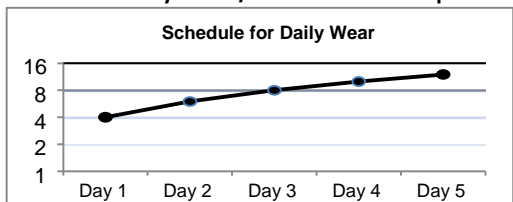

- The lenses are meant for daily wear.
- To be discarded and replaced with new lens every month.
- Always discard lenses worn on a frequent replacement schedule after the recommended wearing schedule by your Eye Care Practitioners.
- Never wear your lenses beyond the period recommended by your Eye Care Practitioners.


Before Use

To prevent infection, please take precautions as follow:

- Check the expired date of the lens before buying.
- Do not use if the sterile package is opened or damaged.
- Use recommended intensive cleaner to wash lens.

Lens Application

- Wash hands thoroughly with soap and dry them with a lint free towel. Rinse the lens thoroughly with multi-purpose solution.
- Place the lens on the tip of your index finger. Make sure the lens is in a bowl shape and correctly oriented.

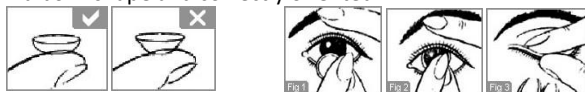

- Hold the upper eyelid firmly with one finger of the other hand. (Fig 1)
- While holding down the lower eyelid with one finger of your applying hand, carefully place contact lens on your cornea. (Fig 2)
- Slowly release upper eyelid, gently close eye to let the lens set in place. Repeat same procedure for the other eye. (Fig 3)

Lens Removal

- Wash hands thoroughly with soap and dry with a lint free towel.
- Check the lens position on cornea.
- Hold down lower eyelid with middle finger.
- Use forefinger to slide lens to lower part of eye.
- Gently pinch the bottom edge of the lens with the pads of index finger and thumb to remove lens.

- Repeat the same procedure for the other eye.

Daily Cleaning and Care Regimen

- After removing the lens from the eye, place it on the palm of your hand and apply a few drops of multi-purpose solution onto the lens.
- Rub the lens gently with your index finger against the palm for about 20 seconds on both sides.
- Rinse the lens thoroughly with multi-purpose solution.
- Soak the clean lens in the case with multi-purpose solution for next use.
- Never rinse lens with tap water.

Contraindications / Warnings / Precautions

- Do not use the contact lens if any of the following conditions exist:
 - Eye disease, injury or abnormality that affects the cornea, conjunctiva, or eyelid
 - Any active corneal infection (bacterial, fungal, protozoal or viral)
 - Eye red or irritated
- Proper use and care of contact lens and lens care products, including lens cases, are essential for the safe use of these products.
- If you experience eye discomfort, excessive tearing, vision blur, redness of the eye or other problems, you should remove your lens immediately and contact your Eye Care Practitioner.
- Always keep your lens immersed in a recommended storage solution when lenses are not being worn (stored). Prolonged periods of drying will reduce the ability of the lens surface to return to a moist state.
- Ask your Eye Care Practitioner about wearing lenses during sporting activities.
- Put on lenses before putting on makeup. Do not get cosmetics, lotions, soaps, creams, deodorants, or sprays in eyes or lenses.
- Do not share contact lenses with others.

Symbol

| | | | |
|--|---------------------|--|-----------------------------|
| | Date of Manufacture | | Steam Sterilized |
| | Lot Number | | Manufacturer |
| | Use By | | EU Representative |
| | Catalog Number | | Consult Instruction For Use |

Manufacturer:

Dreamcon Co., Ltd
10, Yeongdong 2-gil, Yangsan-si, Gyeongsangnam-do, Korea.
Tel: +82 (55) 381-0300 Fax: +82 (55) 366-0211

Principle: One O One Vision Creations Sdn. Bhd. (830877-A)

2nd Floor, Sublot 11, Survey Lot 1529, Block 17,
Jln Lapangan Terbang, 93250Kuching Sarawak Malaysia.
Tel: +6 082-383 101 Fax: +6 082-383 102